

# STANDARD OPERATING PROCEDURE

## DECONTAMINATING THE 3M JUPITER POWERED HOOD

### PREPARATION

1. Get Clinell alcohol wipes (red packet)
2. Put on PPE – gloves/apron/surgical facemask/eye protection
3. Prepare to clean the device in the amber zone, have clinical waste bin to hand

### CLEANING

1. Start by disassembling the device into helmet, tubing and powerpack. Remove the battery and wipe with Clinell alcohol wipe, avoiding the electrical contacts.
2. With Clinell alcohol wipes, wipe over the external surfaces of the powerpack, including the filter covers and waistbelt and buckle.
3. Place the powerpack on a solid surface in the green area.
4. Clean the hosepipe with Clinell alcohol wipes, working into the corrugated pipework.
5. Place the cleaned tubing onto a solid surface in the green area.
6. Clean the plastic helmet with Clinell alcohol wipes, paying particular attention to the hose connector.
7. Place the cleaned helmet onto a solid surface in the green area.
8. Dispose of all wipes into the clinical waste bin.

### AFTER CLEANING

9. Remove PPE and perform hand hygiene.
10. Place the Jupiter into storage, ensuring the batteries are on charge.

**Please note this process will decontaminate (remove the bulk of contamination) the item, but is not a complete disinfection or sterilisation process. After handling the Jupiter, even after decontamination, perform hand hygiene (alcohol gel or hand wash).**